



## Reciprocating File System

### Directions for Use

1. CleanCut Reciprocating files contain nickel and should not be used for patients with a known allergy to this metal.
2. CleanCut files should only be used with a reciprocating handpiece.
3. The CleanCut Reciprocating File System is designed as part of a single use protocol. Once a file has been used, do not reuse it on a different patient.
4. After use, files should be disposed of in an appropriate Sharps container.
5. CleanCut Reciprocating files are available in both sterile and non sterile packaging. Non sterile files must be sterilized before use on a patient. Use standard instrument sterilization protocols with manual or ultrasonic precleaning. Rinse thoroughly and allow to air dry. Place the files in an autoclave tray and run for a 3 minute cycle at 134°C.