**PULPDENT®**

# Etch-All™

## Dental Etching Gel

• 10% H<sub>3</sub>PO<sub>4</sub>

 4 x 1.2 mL Etch-All

Rx only

 **PULPDENT Corporation** 80 Oakland Street • Watertown, MA 02472 • U.S.A.  
[www.pulpdent.com](http://www.pulpdent.com) • [pulpdent@pulpdent.com](mailto:pulpdent@pulpdent.com)

PULPDENT®

Etch-All™  
Dental Etching Gel

**PULPDENT®**

Dental Etching Gel • For Acid Etching Enamel and Dentin

### Etch-All™

#### 10% PHOSPHORIC ACID

Etch-All is a non silica, thixotropic phosphoric acid etch gel which stays in place and will not run onto exposed tissue. Dispensed from a syringe through pre-bent small gauge needles, it offers precise placement of gel even in relatively inaccessible areas. Etch-All washes off easily with water.

#### DIRECTIONS FOR USE

1. Cleanse the enamel surface to be etched with a slurry of pumice or non-oil containing prophylactic paste. Rinse and dry thoroughly.
2. Securely attach 25-gauge luer lock applicator tip to syringe.
3. Apply Etch-All directly onto tooth surface to be etched. Leave in place for 30 seconds while agitating gently.

4. Rinse off Etch-All with copious amounts of water.
5. Leave surface moist or dry surface with clean, uncontaminated air, depending on technique.
6. Re-cap syringe immediately after use.

#### WARNING:

If Etch-All contacts skin or soft tissue, rinse off immediately with copious amounts of water.

**Note:** Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to:

<http://www.fda.gov/MedicalDevices/ProductsandMedicalProcedures/DentalProducts/ucm404472.htm>

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